


OASIS[®]

Shampoo Cap

Instructions for use


Oasis[®] Shampoo Caps are pre-impregnated with a mild shampoo and conditioner. The massaging action releases shampoo and conditioner into the hair and hydrophilic (water-loving) fabric absorbs oil and dirt retaining it within the cap.

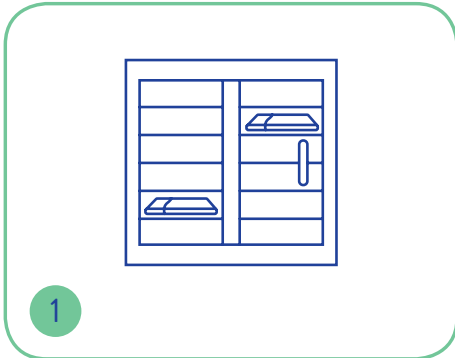


1 Remove shampoo cap from the dedicated Oasis Warmer




2 Or use at room temperature

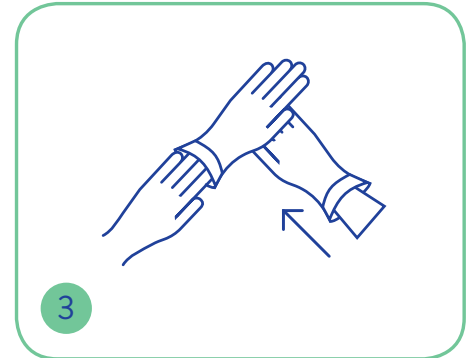


3 Apply protective gloves

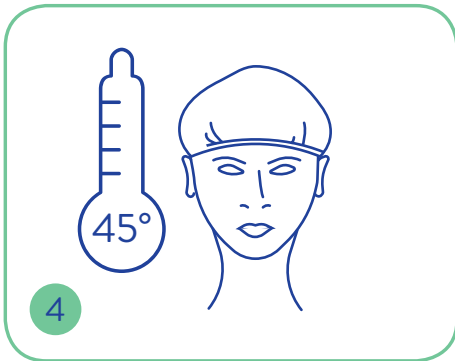

4 Check temperature prior to placing on the user's head, covering all of their hair

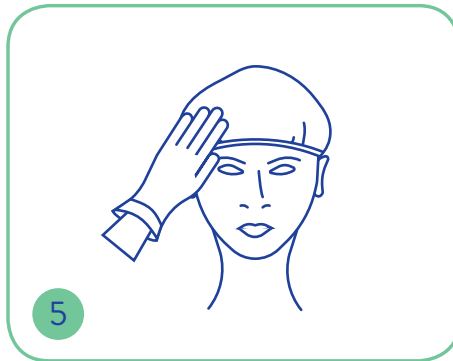


5 Massage thoroughly until hair is saturated

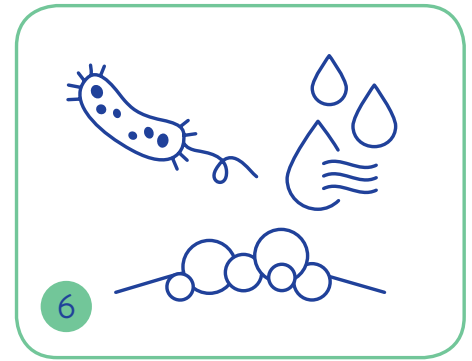

6 The shampoo cap will cleanse and condition the hair



7 Remove the shampoo cap, and dry hair using a towel if excessively wet

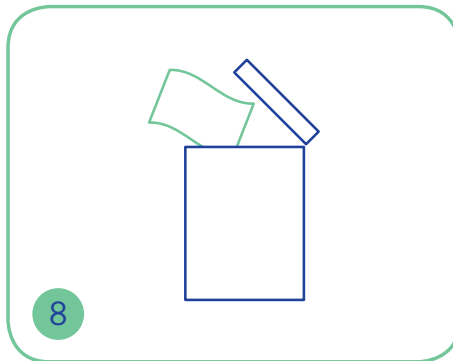

8 Dispose of used shampoo cap

Description	Pack Qty	Case Qty	Code
Oasis [®] Shampoo Cap (Economy)	1	30	RSC965
Oasis [®] Shampoo Cap (Economy) Unscented	1	30	RSC965UP



Perfumed and unperfumed options available